

# Cutimed® PROTECT

Medical skin protection



**Protect  
Preserve  
Prevent**



## Ordering information

Cutimed® PROTECT is available in foam applicators and spray bottle

Cutimed® PROTECT	REF No.	Size	Unit of Measure	HCPCS
<b>Spray</b> For fast, precise, 'no touch' application.	7265300	28 ml / .95 oz.	1	A6250
	7265301	28 ml / .95 oz.	12	A6250
<b>Foam Applicator</b> For fast, clean and precise application to wound margins and peri-wound skin.	7265400	1 ml / .03 oz.	5	A6250
	7265401	1 ml / .03 oz.	25	A6250
	7265402	3 ml / 1.1 oz.	5	A6250
	7265403	3 ml / 1.1 oz.	25	A6250



### Suggested HCPCS Codes

HCPCS Codes listed are only suggested codes. Other codes may be applicable depending on product use. Codes are subject to change without notice

For more information on the Cutimed® range of products please visit us at [www.cutimed.com](http://www.cutimed.com)

*Please refer to the product label and/or package insert for full instructions on the safe use of these products.*



▶ **Vital support for skin at risk.**

Whether managing chronic wounds, incontinence or both, protecting fragile skin may improve outcomes, dressing performance, patient comfort and sense of well being. Skin protection may speed wound healing and reduce dressing change time and discomfort by maintaining a healthy dry interface resistant to adhesives, moisture, friction and shear. Medical Skin Protection goes beyond the wound bed to insure skin surrounding the wound maintains its best possible condition while providing an environment conducive to repair of previously damaged areas. Medical Skin Protection helps maintain the progress you have made towards improving wound and incontinence management and removes barriers to skin and wound healing.

## Understanding threats to the skin's integrity.

The skin is the body's largest organ. Skin regulates temperature, provides sensation, regulates transdermal water loss and provides the barrier between us and the many threats in the external environment. To help maintain healthy function skin requires protection from exudate, incontinence, friction and shear that can break down its natural protective ability. Elderly patients are especially susceptible to external skin stressors including skin stripping from adhesive dressing removal as elderly skin may lose cohesion and become increasingly dry and fragile.



Intact peri-wound skin

### ▶ Risk factors impacting skin at risk:

- | External   | Internal  |
|--|---|
| <ul style="list-style-type: none"><li>• wound exudate</li><li>• urinary or fecal incontinence</li><li>• removal of adhesive dressings</li><li>• friction and shear</li><li>• digestive fluids (ostomy)</li></ul> | <ul style="list-style-type: none"><li>• age</li><li>• nutrition status</li><li>• hydration status</li><li>• ambulatory status</li><li>• systemic disease (diabetes, vascular disease)</li></ul> |



Macerated wound margins

### ▶ Maceration – when the skin's barrier function fails.

Over hydration can lead to maceration of the skin including peri-wound skin and wound margins that can lead to the development or enlargement of wounds. Maceration reduces the skin's ability to resist damage from friction, shear, adhesive dressing removal or continuing damage from incontinence or exudate.

# Medical skin protection

**NEW**

## Cutimed® PROTECT spray and foam applicator: Re-establishing the barrier function of the skin.

All phases of the healing process can benefit from the skin protection provided by Cutimed® PROTECT. It provides a long lasting protective barrier film that shields at risk skin from external factors working against skin and wound healing wherever they may be present. From mechanical stressors such as friction, shear and dressing removal to chemical factors such as exudate and incontinence, skin remains protected and free from irritants.



▶ **Protects wound margins and peri-wound skin**

from external factors including wound exudate, incontinence, friction, shear and adhesive removal

▶ **Preserves and maintains intact skin**

provides a protective breathable film cover to prevent further damage and encourage healing

▶ **Prevents maceration and peri-wound skin breakdown**

by providing up to 96 hours of moisture barrier action

▶ **Promotes dressing adhesion and pain free dressing removal**

Cutimed® PROTECT provides an efficient foundation for adhesives and a protective interface to shield skin and nerves during adhesive dressing or tape removal

▶ **Cutimed® PROTECT foam applicator**

Available in both 1 and 3 ml sizes, Cutimed® PROTECT foam applicators allow clean and precise application of the film to wound margins and peri-wound skin. Cutimed® PROTECT reduces the risk of maceration and isolates the skin from the trauma of adhesive dressing removal, friction and shear.

▶ **Cutimed® PROTECT spray**

Cutimed® PROTECT spray applicator speeds application to larger areas requiring protection from external stressors. The spray applicator assures pain-free 'no touch' delivery to sensitive areas and the efficient application of protection in a cost and time effective manner.

### Used for:

Protection of intact or stressed skin at risk from

- exudate
- incontinence
- friction and shear
- adhesive dressing removal
- digestive enzymes (ostomy)
- maceration

### Properties:


When applied to the skin fast drying Cutimed® PROTECT forms a colorless, transparent, long lasting barrier film. Cutimed® PROTECT is permeable to water vapor and impermeable to liquid water, exudate or incontinence. The no sting Cutimed® PROTECT formula is non-irritating, and alcohol free.

# Medical skin protection




## Cutimed® PROTECT: the skin barrier with proven efficacy.

### ▶ Durable and long lasting – 96 hours of proven protection.

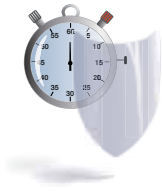


Cutimed® PROTECT has been shown to maintain its barrier function for at least 96 hours against skin staining reagents. Cutimed® PROTECT will be there to shield against moisture and mechanical stress.

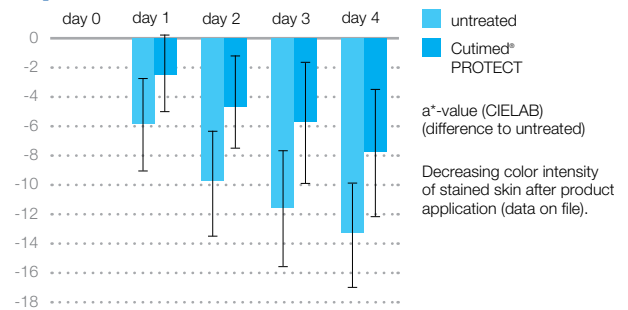

### ▶ Adhesive dressing adhesion.



Adhesive dressing applied to Cutimed® PROTECT treated skin adhere better and stay on longer. At dressing removal time, Cutimed® PROTECT prevents skin stripping and pain associated with adhesive removal.

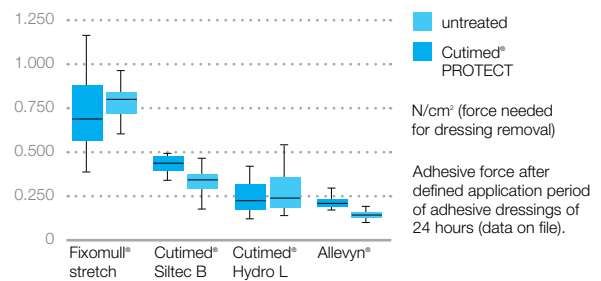

### ▶ Non occlusive protection.



Cutimed® PROTECT regulates transdermal water loss and maintains a healthy level of skin hydration. Cutimed® PROTECT creates a protective film while maintaining high MVTR and gas (oxygen) permeability necessary for skin functions.

