

Cutimed® Hydro

Hydrocolloid Dressings for Managing
a Moist Wound Environment

What is the hydrocolloid of choice for protecting and managing light to moderately exuding wounds?

Cutimed® Hydro:
Protect the wound,
promote healing,
provide comfort



Optimizing a Moist
Wound Healing
Environment

Cutimed® Hydro – focused wound care management




Cutimed® Hydro is a range of sterile, self-adhesive hydrocolloid dressings including Cutimed® Hydro L, mainly for dry to lightly exuding wounds, and Cutimed® Hydro B for light to moderately exuding wounds.

Hydrocolloids absorb exudate to form a gel, which is removed in one piece with the dressing. Cutimed® Hydro dressings promote healing by creating and controlling moist wound conditions.

Cutimed® Hydro – mode of action

The absorbent and expandable particles in the hydrocolloid layer swell as they absorb exudate and form a gel.

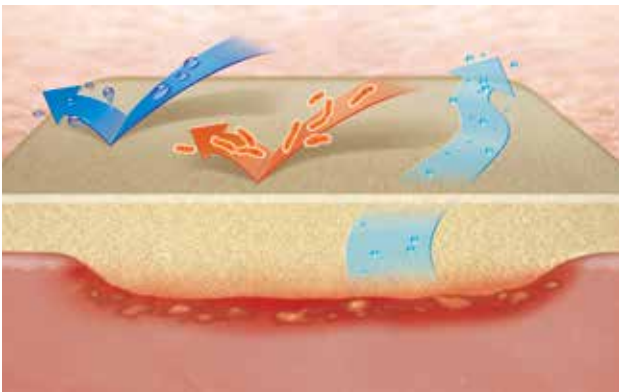

- **Facilitates moist environment**
 - promotes formation of granulation tissue
- **Forms gel in contact with exudate**
 - molds itself to structure of wound
- **Vertical absorption of exudate**
 - keeps wound edges dry and reduces risk of maceration
- **Seals the wound**
 - minimizes contamination from external fluids or bacteria
- **Product structure allows removal of the gel with the dressing**
 - helps protect newly forming tissue

Cutimed® Hydro – user benefits

- **Wide range of dressing shapes and sizes**
 - easy application
- **Smooth and flexible dressing**
 - conforms easily to skin and body contours
- **Intelligent dressing surface**
 - adheres securely to healthy skin, gels on wound





Cutimed® Hydro – patient comfort

- **Retains exudate in gel**
 - reduces leakage risk
 - keeps patient as active as possible
- **Waterproof dressings**
 - can be worn in the shower
- **Rapid dressing changes**
 - minimal patient discomfort
- **Free of latex and gelatin**
 - low allergy risk for patient and clinician



Cutimed® Hydro range

Hydro L is for ‘Light’, because the top layer of semi-permeable, low-friction polyurethane film allows close monitoring of the wound.

Hydro B is for ‘Border’, because the edges of the dressing are tapered for superior adhesion and minimal leakage. The top layer of the dressing is a film-coated polyurethane foam.

Indications

Cutimed® Hydro L is for dry to lightly exuding, acute wounds, such as:

- | | |
|-------------------|---------------------------------------------------|
| ■ Lacerations | ■ Minor burns (superficial and partial thickness) |
| ■ Minor abrasions | ■ Post-op protection of closed surgical sites |



Cutimed® Hydro B is for light to moderately exuding, chronic wounds, such as:

- | | |
|------------------|------------------|
| ■ Venous ulcer | ■ Pressure ulcer |
| ■ Arterial ulcer | ■ Diabetic ulcer |



NOTE: **Cutimed® Hydro L** can also be used as semi-occlusive secondary dressing.

Cutimed® Hydro

- Maintains a moist environment
- Stimulates granulation
- Retains exudate in gel
- Waterproof and breathable

Cutimed® Hydro L			
Size (inches)	REF. No.	Dressings/Box	HCPSC
2 x 10	7263600	12	A6235
3 x 3	7263601	10	A6234
4 x 4	7263602	10	A6234
6 x 6	7263603	5	A6235

Cutimed® Hydro B			
Size (inches)	REF. No.	Dressings/Box	HCPSC
3 x 3	7263500	5	A6237
4 x 4	7263501	5	A6237
6 x 6	7263502	5	A6238
Sacral 6 x 7	7263503	3	A6238

Wound Phase

■ Necrotic

■ Infected

■ Sloughy

■ Granulation

■ Epithelization



Wound Depth

■ Shallow

▼ Deep

▼ Shallow + deep

Exudate

■ Dry to low

■ Low to medium

■ Medium to high



Please refer to the product label and / or package insert for full instructions on the safe use of these products.

BSN medical Inc. 5825 Carnegie Blvd. Charlotte, NC 28209-4633
 Tel. (+1) 704 554 9933 Fax (+1) 704 358 4558 www.bsnmedical.com
 To order toll-free: BSN medical (+1) 800 552 1157

