

# ONE DRESSING FOR BOTH DRY AND MODERATELY EXUDATING WOUNDS

Cutimed® HydroControl –  
An innovative combination of  
a hydropolymer gel matrix and  
a highly breathable PU film

Aiming to effectively treat dry and low to moderately exuding chronic wounds healing by secondary intention? Looking to clean a wound, absorb exudate and create an optimally moist wound environment?

**Cutimed® HydroControl** masters all those challenges in just one dressing.



## Performance Comparison


Comparing the key characteristics of hydrocolloids and thin foam dressings with **Cutimed® HydroControl**, this new form of wound care demonstrates its innovation and performance potential.

Product characteristics	Hydrocolloids	Thin foam dressings	Cutimed® HydroControl
 Excellent exudate management for reduced dressing changes: <ul style="list-style-type: none"> <li>• high absorption capacity</li> <li>• vertical absorption</li> <li>• high moisture vapour transmission rate (MVTR)</li> </ul>	• • •	• • • • • • •	• • • • • • • • •
 Optimal moisture balance	•	• •	• • •
 Facilitate autolytic debridement	•	–	• • •
 Suitable for highly contoured wound locations	•	•	• • •
 Highly adaptable application (thin, flexible, cuttable)	• •	• •	• •

# Cutimed® HydroControl – Focusing on Everyday Use

**Cutimed® HydroControl** is an innovative new product. It supports optimal wound healing conditions by cleaning devitalized tissue while excess exudate is drawn away from the wound bed.<sup>1</sup>

## Application

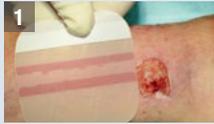

1. Peel away the lower transparent carrier to expose the adhesive surface.



2. Hold the dressing by the white applicator tab and apply with the adhesive surface towards the wound.



3. Carefully apply the adhesive side of **Cutimed® HydroControl** to the wound using light pressure. Then peel away the white tab and press the dressing to the wound.



4. Pull one half of the red tab slowly away from the center of the dressing (in parallel to the skin) and discard. Repeat the last step with the second half of the red tab and discard.

**Tip:** For additional fixation and security during washing/showering we recommend window-framing the dressing with **Hypafix® Transparent**.

## Application to highly contoured wound locations

**Cutimed® HydroControl** adapts to anatomically complex body sites. If necessary the product can easily be cut to size and therefore ensures multiple customized ways of application.



Very good adaptability to the knee joint, even with everyday motion and strain.



Easily adaptable and flexible. For example use on the palm of the hand.



**Tip:** Use **Cutimed® HydroControl** for an effective treatment of a diabetic foot wound. We recommend the use of the smallest size for treating toes.



Cutimed® HydroControl

## Visible success

**Cutimed® HydroControl** has high wound stimulation potential and provides a moist wound healing environment. Therefore it is especially suitable for wounds with poor healing tendency. The example shows a bilateral venous leg ulcer on the outer aspect of the lower leg just above the ankle. The wound was present for 19 months before using **Cutimed® HydroControl**.



Problematic wound before the application of **Cutimed® HydroControl**.



Advanced wound healing progress with **Cutimed® HydroControl** after 4 months<sup>1</sup>.

## Ordering information

Size	Ref. no.	Dressing/Box	Suggested HCPCS Code
1 3/4 x 1 3/4 in	7323100	10	A6234
3 x 3 in	7323101	10	A6234
4 x 4 in	7323102	10	A6234
6 x 6 in	7323103	5	A6235

<sup>1</sup> B. v. Hallern, B. Struensee, Challenges posed by chronic and secondary healing wounds: Cutimed® HydroControl with its new hydropolymer technology provides an innovative therapeutic approach. MEDIZIN & PRAXIS »DIABETIC FOOT« March 2016

### REIMBURSEMENT DISCLAIMER

This document includes codes that might be used to bill for BSN medical Inc. products and related services. Each provider will have to verify the appropriate codes for each patient. It is the provider's sole responsibility to determine and submit appropriate codes, charges, and modifier for services rendered. Providers should contact insurers to verify correct coding procedures prior to submitting claims related to any product or device. BSN medical Inc. cannot guarantee coverage or reimbursement with the codes listed in this billing guide. In all cases, providers will need to follow local payer policies for billing and reimbursement.

Please refer to the product label and/or package insert for full instructions on the safe use of these products.